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Intended use and indication

Our toilet seat raiser is for people for whom the seat height of the toilet is too low due to their height or motor limitations.

Contraindication

When used correctly, there is no known contraindication.

General safety instructions

The toilet seat raiser is intended for use only in connection with standard toilets. It can be mounted on toilets for wall and floor mounting. It is assumed that the toilets intended for mounting the seat raiser are securely fastened. The toilet seat raiser may be used only for toilets on which a safe fixing of the seat raiser is ensured (no camping toilet).

The toilet seat raiser may only be used when mounted. Read the assembly instructions of the toilet seat raiser carefully! If you notice any defects during the use of the toilet seat raiser, remove the seat raiser from the toilet immediately.

 **Attention!**

- Use the toilet seat raiser only for its intended purpose. It is not a substitute for ladders, gymnastics equipment or playground equipment!
- The toilet seat raiser is not intended for examinations or treatments.
- Only use the toilet seat raiser if it is in perfect technical condition. Remove damaged toilet seat raisers from the toilet immediately.
- Only use spare parts and accessories approved by the Kowsky company.
- Do not make any design changes that have not been approved by the manufacturer.
- The toilet seat raiser must not be used in environments where flammable gases or vapors (e.g. anesthetics) are to be expected.

Monatge

Loosen the screws on both sides of the toilet seat raiser until there is enough space for it to be placed on the toilet bowl.

Place the lid and the seat of the existing toilet in an upright position. Then place the toilet seat raiser on the toilet bowl. Make sure that the front lip is directly against the bowl. Now alternately tighten the screws clockwise until the seat has a secure hold on the toilet bowl.

Before each use, make sure that the toilet seat raiser is securely fastened.

 **Attention!**

- Please handle the locking screws with care. Do not continue turning as soon as resistance is noticed.
- Ensure that the toilet seat is mounted horizontally. If mounted at an angle, there is a risk of slipping.

Storage and care

Do not use sharp-edged tools (knives, metal spatulas) or aggressive solvents for cleaning. To avoid damage to health and to the component surfaces, be sure to observe the application instructions for the cleaning agent and disinfectant. Only suitable agents and procedures according to the lists of the German Society for Hygiene and Microbiology (DGHM) may be used for disinfection. Sterilization of the toilet seat raisers is not intended. Light soiling and dust are best removed with a damp cloth. Coarse soiling can be removed with mild cleaning agents free of scouring agents.

Maintenance

The toilet seat raiser is maintenance free.

Limitation of use

The toilet seat raiser may only be loaded according to the manufacturer's instructions. Observe the maximum permissible body weight indicated on the product! For greater loads or in cases of doubt, please contact your specialist dealer. Damaged toilet seat raisers must not be used under any circumstances. **Note:** All serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Reuse (rental) / duration of use

The toilet seat raiser is intended for reuse. Before reuse, it must be cleaned and disinfected. Please also observe the respective lists of the German Society for Hygiene and Microbiology (DGHM). The product service life is set by the manufacturer at three years if used as intended. Beyond this time, the continued use of the toilet seat raiser is the responsibility of the user.

Warranty and liability

You have purchased a high-quality Kowsky product. If, despite the highest quality standards, a justified material defect should occur, Kowsky has the right, within the scope of a 12-month warranty, to repair the defective item or to replace it with a defect-free subsequent delivery, at its own discretion. We cannot accept any liability for damage caused by modifications to our products. Defects which are due to wear and tear or improper handling are not covered by the warranty. Wear parts are excluded from the warranty. Only original spare parts and accessories may be used! For spare parts, accessories and repairs, please contact the specialist trade.

Disposal

Please ask your specialist dealer about the regional disposal regulations.

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Item number	100
Name	Toilet seat raiser with lid
Width	36,9 cm
Depth	40,7 cm
Height front	10 cm
Height back	13 cm
Opening dimensions	27 cm x 21,5 cm
Weight	1,3 kg
Max. load capacity	160 kg

Symbol directory

 REF Item number	 LOT Batch number	 Read the operating instructions	 ! Attention	 This product complies with the requirements of Regulation (EU) 2017/745 (MDR).
 Manufacture	 Date of manufacture	 Permissible body weight	 MD Medical device	

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It is a product in accordance with the guidelines of the Medical Devices Directive. Furthermore the Erwin Kowsky GmbH & Co. KG is certified according to DIN EN ISO 13485.