DE

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EN Toilet seat raiser with armrests and lid

The toilet seat raiser is intended for use by people for whom standard toilets are too low. Before using the toilet seat raiser, consult your doctor to determine whether it is the right aid for you. The following instructions provide you with descriptions of how to assemble, operate and care for the toilet seat raiser.

If you have any questions, please contact your medical supply store or Kowsky directly.

You can reach us at the following address:

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DIN EN ISO 13485
REG.-NR. MQ1 0115318

It is a product in accordance with the guidelines of the Medical Devices Directive.
Furthermore the Erwin Kowsky GmbH & Co. KG is certified according to DIN EN ISO 13485.

1.0	Indication	10
2.0	Contraindication	10
3.0	General safety instructions	10
4.0	Scope of delivery/parts designation	11
5.0	Mounting the toilet seat raisers	11
6.0	Mounting the armrests	12
6.1	Inserting the armrests	12
6.2	Removing the armrests	12
7.0	Opening the toilet seat raiser	12
8.0	Adjusting the seat height	12
8.1	Required tools	13
8.2	Adjusting the mounting tube	13
8.3	Adjusting the support tube	13
9.0	Cleaning and disinfection	13
10.0	Maintenance	14
11.0	Re-use	14
12.0	Warranty and liability	14
13.0	Disposal	14
14.0	Technical data	14
15.0	Symbol directory	15

1.0 Indication

Our height-adjustable toilet seat raiser with armrests and lid is suitable for people with a maximum body weight of up to 120 kg. The seat raiser facilitates toilet use for persons with difficult toilet use due to inappropriate seat height due to restricted ability to sit down and stand up and restricted upright sitting position in case of dysfunction of the lower extremities due to restricted or completely suspended mobility of the joints.

2.0 Contraindication

There is no known contraindication when used correctly.

3.0 General safety instructions

The toilet seat raiser is only intended for use in connection with commercially available toilets. It can be mounted on toilets for wall and floor mounting. It is assumed that the toilets intended for the installation of the seat raiser are securely fastened. The toilet seat raiser may only be used for toilets on which a secure fastening of the seat raiser is guaranteed (no camping toilet).

The toilet seat raiser may only be used when assembled. Read the assembly instructions for the toilet seat raiser carefully! If you notice any defects while using the toilet seat raiser, remove it from the toilet immediately.

Attention!

- Use the toilet seat raiser only for its intended purpose. It is not a substitute for ladders, gymnastics equipment or playground equipment!
- The toilet seat raiser is not intended for examinations or treatment.
- Only use the toilet seat raiser when it is in technically perfect condition. Remove damaged toilet seat raisers from the toilet immediately.
- Only use spare parts and accessories approved by Kowsky.
- Do not make any design changes that have not been approved by the manufacturer.
- The toilet seat raiser must not be used in environments where flammable gases or vapours (e.g. anaesthetics) are likely to be present.

4.0 Scope of delivery/parts designation

The toilet seat raiser is delivered packed in a carton. Check the contents of the packaging against the list below before you start assembly. In case of visible defects or missing parts, contact the Kowsky company or your medical supply store.

Carton contents:

1 x Toilet seat raiser

...with the following fastening parts:

2 x (1) hand nut (M8)

2 x (2) clamp

2 x (3) threaded rod (M8 x 140 mm)

1 x (4) mounting tube

2 x (5) Hand screw (M8 x 12 mm)

2 x Armrest

5.0 Mounting the toilet seat raiser

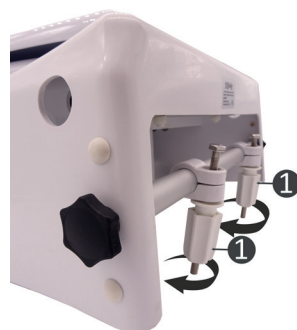
1. Remove the existing toilet seat (follow the installation instructions for the item).
2. Remove the hand nuts (1) from the threaded rods of the of the toilet seat raiser.
3. Adjust the clamps by moving them to the distance of the of the mounting holes of the toilet bowl.
4. Guide the threaded rods through the holes and screw on the hand nut from below.

Note: Make sure that the clamps face to the rear edge of the toilet.

5. Align the toilet seat raiser straight with the toilet and tighten the hand nuts.

Note: Excessive tightening with tools will destroy the hand nut.

Note: After assembly, check whether the toilet lid and the toilet seat can be opened far enough. In order to prevent the toilet seat from closing by itself, it must be possible to open both covers more than 90°.

After mounting, check that the toilet seat raiser is properly seated and firmly fixed!

6.0 Mounting armrests

The two armrests of the toilet seat raiser can be folded up to increase freedom of movement or removed for easier cleaning.

6.1 Inserting the armrests

1. Push the armrest through the side hole (6) onto the guide tube, press in the securing pin (7) and push the armrest in completely.
2. Place the armrest on the support (8).
3. Insert the second armrest as described above.

6.2 Removing the armrests

1. Fold up the armrest from the support (8).
2. Reach into the toilet seat raiser from behind and press in the securing pin (7).
3. Pull the armrest outwards from the guide tube (6).
4. Remove the second armrest as described above.

7.0 Opening the toilet seat raiser

For easy cleaning, the toilet seat raiser can be opened up. Before opening the toilet seat raiser, remove the armrests and loosen both hand screws of the tube counterclockwise. After cleaning, simply put the toilet seat raiser back down and tighten both hand screws of the mounting tube clockwise.

After mounting, check that the toilet seat raiser is properly seated and firmly fixed!

8.0 Adjusting the seat height

The seat height of the toilet seat raiser can be set to three seat heights. To adjust the height, the rear mounting tube (4) can be screwed to three fixing holes for height adjustment. The support tube in the front part (9) of the toilet seat raiser must be adjusted to the height of the mounting tube.

EN Toilet seat raiser with armrests and lid

Actual seat elevation (top edge of toilet bowl - seat surface toilet seat raiser):

- a. top hole 80 mm
- b. middle hole 120 mm
- c. bottom hole 160 mm

8.1 Required tools

- 1 x slot screwdriver
- 1 x ring spanner (spanner size = 13 mm)

8.2 Adjusting the mounting tube

1. Remove the cover cap at the desired height of the corresponding hole.
2. Unscrew both hand screws of the mounting tube by turning them anticlockwise.
3. Align the mounting tube with the holes and screw it in by turning the hand screws clockwise.
4. Press the cover caps into the free holes.

8.3 Adjusting the support tube

Note: The support tube must be screwed in the same hole height as the mounting tube, e.g. mounting tube in middle hole = support tube in middle hole.

1. Carefully remove the caps of the support tube by using the screwdriver.
2. Remove the caps from the holes in which the support tube is to be screwed.

Risk of injury!

Do not hold any part of your body in the piercing direction of the screwdriver.

3. Unscrew the fastening screws by turning them anticlockwise.
4. Align the support tube with the new holes and screw it in place.
5. Replace all caps.

9.0 Cleaning and disinfection

Do not use sharp-edged tools (knives, metal spatulas) or aggressive solvents for cleaning. To avoid damage to health and the surfaces of the components, be sure to follow the instructions for use of the cleaning and disinfecting agent. Only suitable agents and procedures according to the lists of the German Society for Hygiene and Microbiology (DGHM) may be used for disinfection. Sterilisation of the toilet seat raisers is not intended. Light soiling and dust are best removed with damp cloths. Coarse soiling can be removed with mild cleaning agents free of scouring agents.

10.0 Maintenance

The toilet seat raiser is largely maintenance-free, but it should be inspected once a month. If you find any defects during use or testing, remove the toilet seat raiser from the toilet immediately and replace the damaged components.

Spare parts can be obtained from any medical supply store that sells Kowsky products.

Carry out the following checks:

1. Visual inspection of all parts for deformation, damage, wear and corrosion.
2. Check all connecting elements (e.g. nut, screws, etc.) for completeness, function and proper fit..

11.0 Re-use

The toilet seat raiser is intended for reuse. Before reuse, it must be cleaned and disinfected. Please also observe the respective lists of the German Society for Hygiene and Microbiology (DGHM).

12.0 Warranty and liability

You have purchased a high-quality Kowsky product. If, despite the highest quality standards, a justified material defect should occur, Kowsky has the right, within the scope of a 12-month warranty, to repair the defective item or to replace it with a defect-free subsequent delivery, at its own discretion. We cannot accept any liability for damage caused by modifications to our products. Defects which are due to wear and tear or improper handling are not covered by the warranty. Wear parts are excluded from the warranty. Only original spare parts and accessories may be used! For spare parts, accessories and repairs, please contact the specialist trade.

13.0 Disposal

When disposing of the toilet seat raiser, the regional disposal regulations must be observed. The metal parts such as the armrests, the support tube and the support tube must be separated from the toilet seat raiser and handed over for recycling. The plastic parts can be taken to the appropriate plastics recycling facility.

14.0 Technical data

Max. load 120 kg
weight 6 kg

Dimensions:

Total height without armrests 20 cm
Total height with armrests 36 cm
Total width without armrest 46 cm
Total width with armrests 61 cm
Total depth 50 cm

15.0 Symbol directory

REF

Item number

LOT

Batch number

Read the operating instructions

Manufacture

Date of manufacture

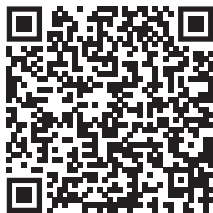
Attention

Permissible body weight

This product complies with the requirements of Regulation (EU) 2017/745 (MDR).

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