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Instructions for Use: Five-foot crutch	EN
Intended use and indication	Walking aids are designed for people with significant mobility impairment with impairment to movements, compromised balance and stability with sufficiently intact walking ability and coordination. They are used to maintain, promote and stabilise walking or to partially relieve the strain on the lower limbs. The walking aids can be used both indoors and outdoors. It is imperative that the safety instructions are also observed!
Contraindicatio n	The use of walking aids is unsuitable if there is a loss of limb in both arms or joint contractures/joint damage in both arms.
General safety information	Do not hang any bags, pouches or other objects on the walking aid! Walking aids may under no circumstances be used for any other purpose than that for which they are intended, e.g. for lifting or moving objects or as a tool for switching light switches or pressing buttons on lifts! It is essential that the ground has a good grip to ensure a secure hold of the walking aids! Avoid wet floors or surfaces such as gravel, sand, foliage, ice or snow! Pay attention to stumbling points such as unevenness, holes in the floor, loose floor coverings or edges, because the walking aid can unexpectedly slip away or sink in! Extra caution is recommended when lighting conditions are insufficient, since irregularities in the ground can then only be seen with difficulty! The holes for adjusting the walking aid undergo a certain amount of wear. It is important to ensure that they are not worn out, as this would prevent secure locking. The plastic clip must have a positive fit when inserted (Fig. 1)] Prior to each use, make sure that all screw connections are firmly tightened!

Adjusting the walking aid
Warning! Please contact your specialist dealer to ensure proper functional and ergonomic adjustment! Incorrectly adjusted walking aids may restrict the functionality of the walking aid or lead to corrosion forming and consequently to an increased risk of breakage. Avoid prolonged contact of the walking aids with moisture. The walking aids must be dried appropriately when they come into contact with water. The correct handling of the walking aids takes some practice. We therefore recommend getting additional support from an assistant at the beginning. Always use the walking aids on the side opposite the injured leg or the side that needs to be protected! Guide the walking aids close to the body, not at an angle facing outwards (Fig. 2)] You might fall otherwise! During use, grasp the handle(s) of the walking aid(s) completely and firmly with your hand! Always place the walking aids as straight as possible on the ground so that you do not slip away! Find chairs with armrests to sit on. You will then be able to support yourself on the armrests when standing up! Wear shoes that are as comfortable as possible with flat soles so that your leg can better withstand additional strain! **If you do fall: if possible, let the walking aid fall to the side so that you do not fall onto it! If someone accompany and support you if possible, so that they can catch you if you fall! Try and arrange for qualified personnel to instruct you in climbing stairs with a walking aid!**

Use
There are many different ways to use the five-foot crutch and it all depends on the individual restrictions of the walking ability. Seek instruction in the use of the walking aids from your specialist dealer or health care specialist! Walking aids must not be used in water. Any penetrating water can lead to corrosion forming and consequently to an increased risk of breakage. Avoid prolonged contact of the walking aids with moisture. The walking aids must be dried appropriately when they come into contact with water. The correct handling of the walking aids takes some practice. We therefore recommend getting additional support from an assistant at the beginning. Always use the walking aids on the side opposite the injured leg or the side that needs to be protected! Guide the walking aids close to the body, not at an angle facing outwards (Fig. 2)] You might fall otherwise! During use, grasp the handle(s) of the walking aid(s) completely and firmly with your hand! Always place the walking aids as straight as possible on the ground so that you do not slip away! Find chairs with armrests to sit on. You will then be able to support yourself on the armrests when standing up! Wear shoes that are as comfortable as possible with flat soles so that your leg can better withstand additional strain! **If you do fall: if possible, let the walking aid fall to the side so that you do not fall onto it! If someone accompany and support you if possible, so that they can catch you if you fall! Try and arrange for qualified personnel to instruct you in climbing stairs with a walking aid!**

Precautionary measures
Using a walking aid places an unusual amount of strain on individual parts of the body! This may result in the following undesirable side effects: The hands may encounter pressure or chaffng! Keep in mind that this can lead to pain and overexertion at the beginning or during prolonged exertion! It is important to take sufficient breaks!

Storage and care
Never store the walking aids in the vicinity of heat sources such as furnaces, heaters or in places susceptible to heat in a car! Do not expose walking aids to sunlight for prolonged periods of time! Strong sun or heat damages the plastic! **The rubber buffer is a wearing part and must be replaced at regular intervals.** Walking aids must only be used again after an extended period of non-use following prior inspection by qualified personnel! **Warning!** Never clean handles and rubber caps using oily substances! There is a risk of slipping! In the event of contamination, clean the walking aids using a mild cleaning agent and a soft cloth where necessary. **Never** use cleaners containing solvents, since these will damage the plastic! **Note:** Should disinfection be required, then a suitable disinfectant agent should be used. Please follow the manufacturer's instructions for use.

Limitation of use
The walking aid must only be subjected to loads in accordance with the manufacturer's instructions. **Observe the maximum permissible body weight that is specified on the product!** Please contact your specialist dealer in the case of greater loads or in case of any doubts. Bent or damaged walking aids must not be used under any circumstances, since they can no longer offer the necessary stability! Bent walking aids must also not be straightened because of the risk of breakage! **Note:** All serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is located.

Reuse (rental) / duration of use
The walking aid may only accept liability pursuant to the Product Liability Act for the initial placing of our products on the market. We only consent to reuse if we have first inspected the product. **The service life of the walking aids is limited to 2 years.** The user shall be responsible for the continued use of the walking aids after this time.

Warranty and liability
You have purchased a high-quality Osseberg product. In the event of a justified material defect in spite of the highest quality standards, Osseberg reserves the right, as part of a 12-month warranty, to either repair the defective item or replace it with a new item of the same type, at its own discretion. Please note that strict legal regulations are applicable to our finished products, in particular custom-made products. We shall not be liable for any damage caused by modifications to our products. The warranty does not cover defects attributable to wear and tear or improper handling. Wear parts are excluded from the warranty. Only original replacement parts and accessories must be used! Please contact your specialist dealer for replacement parts, accessories and repairs.

Disposal
Please consult your specialist dealer about the regional disposal regulations.

Technical data					
REF	A	B	C		
17	Five-foot crutch	79 cm- 94 cm	21 cm	10 mm	100 kg
Replacement parts					
REF					
17	Five-foot crutch	Art. 131620		Art. 910	

Accessories					
The approved accessories for your walking aid can be found at: https://shop.kowsky.de/downloads-zum-produkt/					
Symbol directory	REF	Item number	LOT	Batch number	
	Attention		This product complies with the requirements of Regulation (EU) 2017/745 (MDR)		Manufacturer
	Permissible body weight		Date of manufacture		Date of manufacture
	Manufacturer		Permissible body weight		Date of manufacture
	Manufacturer		Permissible body weight		Date of manufacture

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