

**Intended use and indication**

Our rubber capsules are used to maintain, promote and secure walking in conjunction with a walking aid or walking stick.

**Please also be sure to read the safety instructions!**

**Contraindication**

Not available.

**General safety information**

Before each use, check your walking aid for proper functioning and a tight fit of the rubber capsule. Check the rubber capsule regularly for damage or wear and replace it if necessary. The grip of the underground is crucial for the safe hold of the walking aids! Avoid wet floors or surfaces such as gravel, sand, leaves, ice or snow! Pay attention to stumbling points like unevenness, holes in the ground, loose floor coverings or edges, because the going assistance can slip away unexpectedly or sink in! Special caution is appropriate with insufficient lighting conditions, since then irregularities in the underground are only badly recognizable!

**Mounting the rubber capsule**

Remove the old rubber capsule from your walking aid or walking stick and put the new rubber capsule on the tube. **Attention!** Be sure to check the tight fit of the rubber capsule.

**Use**

The risk of slipping or falling- especially on slippery and wet surfaces- can never be completely ruled out when using walking aids. When walking, place the crutch or walking stick as vertically as possible and not too far away from the body. A very oblique placement increases the risk of falling!

**Storage and care**

Never store the rubber capsule near heat sources such as ovens, heaters or in the car on the shelf! Do not expose the rubber capsule to sunlight for a longer period of time! Strong sun or heat will damage the material! **The rubber capsule is a wearing part and must be replaced regularly. Warning!** Never clean rubber capsules with oily substances! Danger of slipping! If necessary, clean the rubber capsule with a mild cleaning agent and a soft cloth if it is dirty. Never use cleaners containing solvents, as these will damage the material! **Note:** If disinfection becomes necessary, use a suitable disinfectant. Please note the manufacturer's instructions for use.

**Limitation of use**

Damaged and heavily worn rubber capsules must not be used under any circumstances, as they no longer provide the required safety! **Note:** All serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

**Re-use (rental) / duration of use**

We accept liability in accordance with the Product Liability Act only for the initial placing of our products on the market. We only agree to a reuse if the product has been tested by us beforehand. **The service life of a rubber capsule depends on wear, but is limited to a maximum of 2 years.** Beyond this time, further use of the rubber capsule is the responsibility of the user.

**Warranty and liability**


You have purchased a high-quality Kowsky product. If, despite the highest quality standards, a justified material defect should occur, Kowsky has the right, within the framework of a 12-month warranty, to repair the defective item or to replace it with a defect-free subsequent delivery, at its own discretion. We cannot accept any liability for damage caused by modifications to our products. Not covered by the warranty are defects that are due to wear or improper handling. Wear parts are excluded from the warranty. Only original spare parts and accessories may be used! For spare parts, accessories and repairs, please contact the specialist trade.

**Disposal**

Please consult your specialist dealer about the regional disposal regulations.

**Technical data**

REF		A	REF		A
71600	Rubber capsule with steel inlay	16 mm	71601	Rubber capsule without steel inlay	16 mm
17101	Rubber capsule with steel inlay	18 mm	17130	Rubber capsule without steel inlay	18 mm

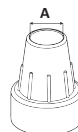

**Symbol directory**

Item number	Batch number	Read the operating instructions	This product complies with the requirements of Regulation (EU) 2017/745 (MDR).
Manufacture	Date of manufacture	Attention	Medical device
Distribution by	Permissible body weight		

**Erwin Kowsky GmbH & Co. KG** Phone +49 (0) 43 21- 99 57 0  
 Oderstraße 73 Fax +49 (0) 43 21- 99 57 57  
 24539 Neumünster Internet www.kowsky.de  
 GERMANY E-Mail info@kowsky.de

